
Vaginal Applicator

EN for use in combination with octenisept®

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DE zur Anwendung mit octenisept®

IT per l'utilizzo con octenisept®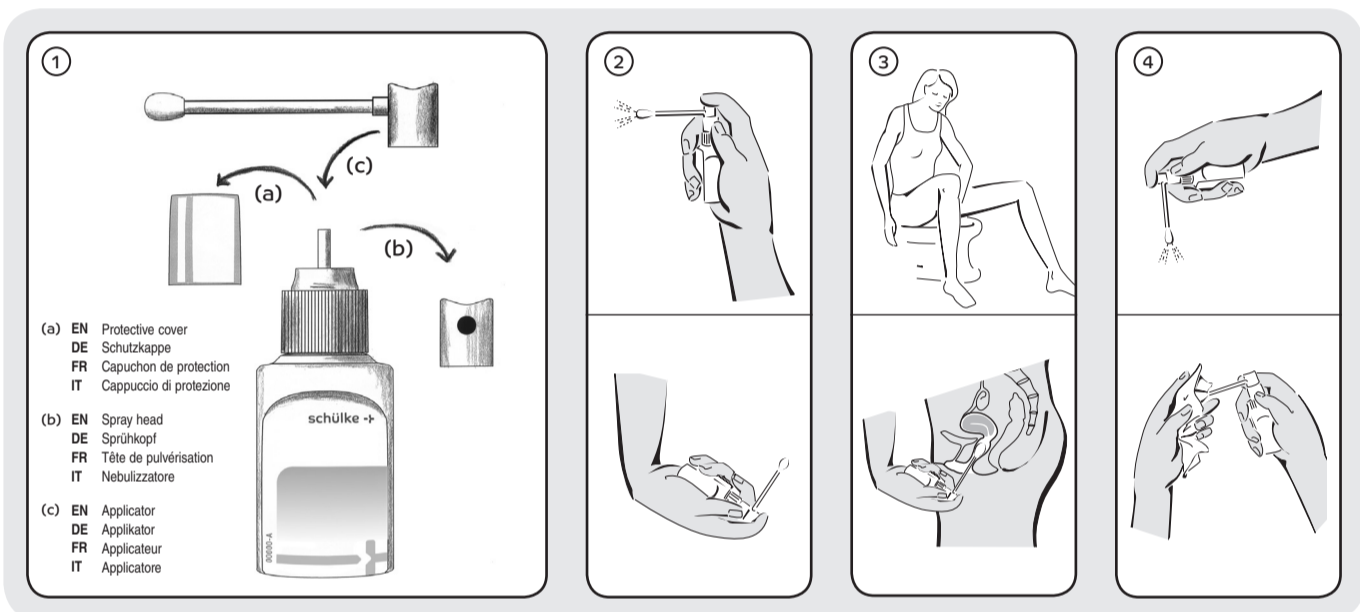

EN Information for use

- Prior to first use, take this applicator (c) out of the wrapper and, after removing the transparent protective cap (a) and spray head (b), place it on the octenisept® bottle.
- Before inserting the applicator, press the spray head several times until the product comes out of the openings of the applicator. The spray mechanism can be operated with the bottle in different positions (upright or upside-down).
- We recommend sitting on the toilet to insert the applicator deep into the vagina. Push the cap of the applicator 10 times using your index finger in order to distribute the product deep within the vagina.
- Hold the applicator downwards and spray some of the product out, for example, over the sink. Afterwards, rinse off the applicator under running water and dry it with a clean cloth.

Before the next use, repeat the cleaning as described under 4. and then perform the treatment as outlined.

The applicator can be used for the duration of a treatment cycle. After the end of the vaginal therapy, the applicator should be discarded in household waste. The applicator is not suitable for patients under the age of 8. The applicator is to be used exclusively for the intravaginal application of octenisept® and it is not suitable for other body orifices (such as the nose, ear, anus).

Please read instructions for use of octenisept® vaginal therapeutic agent.

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Schülke & Mayr GmbH • Robert-Koch-Str. 2 • 22851 Norderstedt
Schülke & Mayr AG • CH-8003 Zürich • Tel. +41 44 466 55 44
Schulke & Mayr UK Ltd. • GB-Sheffield S9 1AT • Tel. +44 114 254 3 500